



MPF BRUSH HYDRATOR

Item No: 109-0000

Ladies & Gentleman MPF Brush Hydrator is back in the market with a brand new design.



*brush not included

Kolinsky hair brushes tend to be difficult to moisten and prepare for build up. This water reservoir suspends the brush without touching the sides or bottom of the vessel.

After 5 minutes of soaking, the water, slowly permeates its way into the hair bundle, giving perfect moisture level required for application.



ATTENTION: The Hydrator is **NOT** to be used as an over night stand to suspend the brush. If the brush is left for a long period of time in water, problems to the hair may occur. The water in the Hydrator should be changed on a regular basis.