Description

Adhese® Universal is a light-curing single-component dental adhesive for enamel and dentin and is compatible with all etching techniques (self-etch, selective-enamel-etch and etch & rinse techniques).

Composition

Adhese Universal contains methacrylates, ethanol, water, highly dispersed silicon dioxide, initiators and stabilizers.

Indications

- Direct-placed light-curing composite and compomer restorations.
- Direct-placed core build-ups with light-, self- and dual-curing composites.
- Repairs of fractured composite and compomer restorations.
- Adhesive cementation of indirect restorations with light- and dual-curing luting composites.
- Sealing of prepared tooth surfaces before temporary / permanent cementation of indirect restorations.
- Desensitization of hypersensitive cervical areas.

Contraindication

- Do not use Adhese Universal if the patient is known to be allergic to any of the materials' ingredients or if the stipulated working technique cannot be employed.
- Applications in which sufficient illumination cannot be ensured (e.g. cementation of root canal posts).
- Direct pulp cappings.

Side effects

In rare cases, components of Adhese Universal may lead to sensitization. In such cases, Adhese Universal should no longer be used.

Interactions

- Peroxide-based bleaching agents and disinfectants may inhibit the polymerization of adhesives. Such materials must not be used ≤ 14 days prior to the application of Adhese Universal.
- Phenolic substances (e.g. eugenol, salicylic derivatives) may inhibit the polymerization of Adhese Universal.
- Iron- and aluminium-based styptic agents must not be used shortly before or during the application of Adhese Universal.
- Fluoridation agents must not be used prior to the application of Adhese Universal.

Procedure for direct and indirect restorations**1. Isolation**

Adequate relative or absolute isolation using suitable auxiliaries, such as OpraGate® or OpraDam® Plus, is required.

2. Pulp protection / cavity liner

In very deep cavities, areas close to the pulp must be selectively coated with a calcium hydroxide liner (e.g. ApexCal®) and subsequently covered with pressure-resistant cement (e.g. glass ionomer cement such as Vivaglass® Liner).

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3. Conditioning with phosphoric acid gel (optional)

The bond to enamel can be improved by selectively etching the enamel or by applying the "etch & rinse" technique. Unprepared enamel surfaces must be conditioned with phosphoric acid (e.g. Total Etch). Please observe the instructions for use for the phosphoric acid gel.

a. Selective-enamel-etch technique

Apply phosphoric acid gel (e.g. Total Etch) onto the enamel and allow it to react for 15–30 seconds. Then rinse thoroughly with a vigorous stream of water for at least 5 seconds and dry with oil- and water-free compressed air until the etched enamel surfaces appear chalky white.

b. Etch & rinse technique

Apply phosphoric acid gel (e.g. Total Etch) onto the prepared enamel first, and then onto the dentin. The etchant should be left to react on the enamel for 15–30 seconds and dentin for 10–15 seconds. Then rinse thoroughly with a vigorous stream of water for at least 5 seconds and dry with oil- and water-free compressed air until the etched enamel surfaces appear chalky white.

4. Handling VivaPen and bottle

a. VivaPen

– Remove the VivaPen cap by turning it until a noticeable resistance is overcome whilst simultaneously pulling it off.

– Attach the VivaPen brush cannula. The cannula can be rotated up to approximately 270° to the desired position.

Attention: Rotating the cannula past the point of resistance will detach the cannula!

– For hygienic reasons, always use a VivaPen protective sleeve by sliding the Pen with the cannula into the protective sleeve until the cannula punctures through the sleeve.

– A new cannula and protective sleeve must be used for each patient.

– Activate the click mechanism 2–3 times until the cannula tip is sufficiently coated with Adhese Universal. The wetting of the cannula tip is recognized by the yellow colour of Adhese Universal.

– If used several times on the same patient, the moistened cannula must be protected from light.

– Remove the protective sleeve and cannula after having used the VivaPen. Remove the cannula by turning it until a noticeable resistance is overcome whilst simultaneously pulling it off.

– Close the VivaPen with the original cap.

Notes

– When using the VivaPen for the first time, more than 2–3 clicks may be required to wet the cannula brush due to the engineering of the VivaPen. This does not influence the indicated fill-level or the function of the VivaPen.

– Do not immerse the VivaPen in disinfecting solutions or autoclave it.

– Contamination on the VivaPen can be removed, e.g. by wiping with alcohol or other non-oxidizing disinfectants.

– For hygienic reasons, VivaPen protective sleeves should always be used.

– Cannulas other than those supplied with the VivaPen are not compatible with the VivaPen and must not be used.

– The fill-level indicator of the VivaPen identifies the remaining amount of adhesive: If the white indicator is no longer visible on the display, the VivaPen is empty.

b. Bottle

– Dispense the desired amount of Adhese Universal into a mixing well and apply it using a disposable applicator (e.g. Vivadent® Applicator Brush). Protect the adhesive from light (e.g. VivaPad®).

– Carefully close the bottle immediately after each use.

– A new disposable applicator must be used for each application.

Notes

– Contamination on the bottle can be removed, e.g. by wiping with alcohol or other non-oxidizing disinfectants.

– Do not use the bottle intraorally. For hygienic reasons, suitable application aids should be used (e.g. VivaPad®).

5. Application of the adhesive

– Starting with the enamel, completely coat the tooth surfaces to be treated with Adhese Universal.

– The adhesive must be scrubbed into the tooth surface for at least 20 seconds. This time must not be shortened. Applying the adhesive on the tooth surface without scrubbing is inadequate.

– Disperse Adhese Universal with oil- and moisture-free compressed air until a glossy, immobile film layer results.

Important: Avoid pooling, since this can compromise the fitting accuracy of the permanent restoration.

6. Light-curing the adhesive

Light-cure Adhese Universal for 10 seconds using a light intensity of ≥ 500 to ≤ 1200 mW/cm².

7. Application of the restorative or luting composite

Proceed according to the instructions for use of the restorative and/or luting composite used.

Procedure for intraoral repairs of fractured composite fillings/composite restorations with light-curing materials

– Roughen the surface to be repaired using diamond finishers and thoroughly clean with water spray and dry with oil- and water-free compressed air.

– Proceed according to the direct/indirect application for Adhese Universal.

Procedure for the conditioning of Tetric® CAD restorations

– Proceed according to the instructions for use for Tetric CAD.

Warnings for Adhese Universal

– Adhese Universal is irritant. Avoid contact with the skin, mucous membrane and eyes.

– In case of skin contact, immediately wash with soap and copious amounts of water.

– If the material should accidentally come in contact with the eyes, immediately rinse with copious amounts of water and consult a physician / ophthalmologist.

– In rare cases, contact with skin may lead to sensitization towards the ingredients.

– Commercial medical gloves do not provide protection against the sensitizing effects of methacrylate.

– Do not use Adhese Universal as a primer for ceramic restorative materials. A suitable ceramic primer must be used (e.g. Monobond Etch & Prime®).

Warnings for phosphoric etching gel

Phosphoric etching gel is corrosive. Avoid contact with the eyes, mucous membrane and skin (protective goggles are recommended for both the patients and practitioners). If the material should accidentally come in contact with the eyes, immediately rinse with copious amounts of water and consult a physician/ ophthalmologist.

Shelf life and storage

– Expiration date: see imprint on packaging.

– Immediately close the VivaPen and/or bottle with the original cap after use.

– Storage temperature 2–28 °C / 36–82 °F.

Keep out of the reach of children!

For use in dentistry only!

The product has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damage resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

The Material Safety Data Sheet is available online at www.ivoclarvivadent.com

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