lay:art Anmischplatten
tropic XL / tropic S / tropic pro XL / tropic pro S

No. 1044 0000 / 1044 1000 / 1045 0000 / 1045 1000

Bedienungsanleitung
Instruction manual • Mode d'emploi
Istruzioni d'uso • Instrucciones para el servicio
Kullanım Kılavuzu
Инструкция по эксплуатации
Instruções de operação • 操作说明书 • 取扱説明書
사용 설명서 • تامي ل عتلا ل ي ل د

≡ lay:art

lay:art Anmischplatten

tropic XL / tropic S / tropic pro XL / tropic pro S

No. 1044 0000 / 1044 1000 / 1045 0000 / 1045 1000

DEUTSCH

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DE

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ENGLISH

Instruction manual

Congratulations, and thank you for choosing the high quality and state-of-the-art porcelain mixing palette system „lay:art“ by Renfert.

EN

1. Description

The system consists of a high quality plastic case with a lid and a porcelain mixing palette of your choice.

1.1 Components and functional elements

- 1 Base
- 2 Lid
- 3 Mixing palette
- 4 Labelling foil
- 5 4 x Case feet
- 6 4 x Rest plugs (short)
- 7 Non-permanent marker

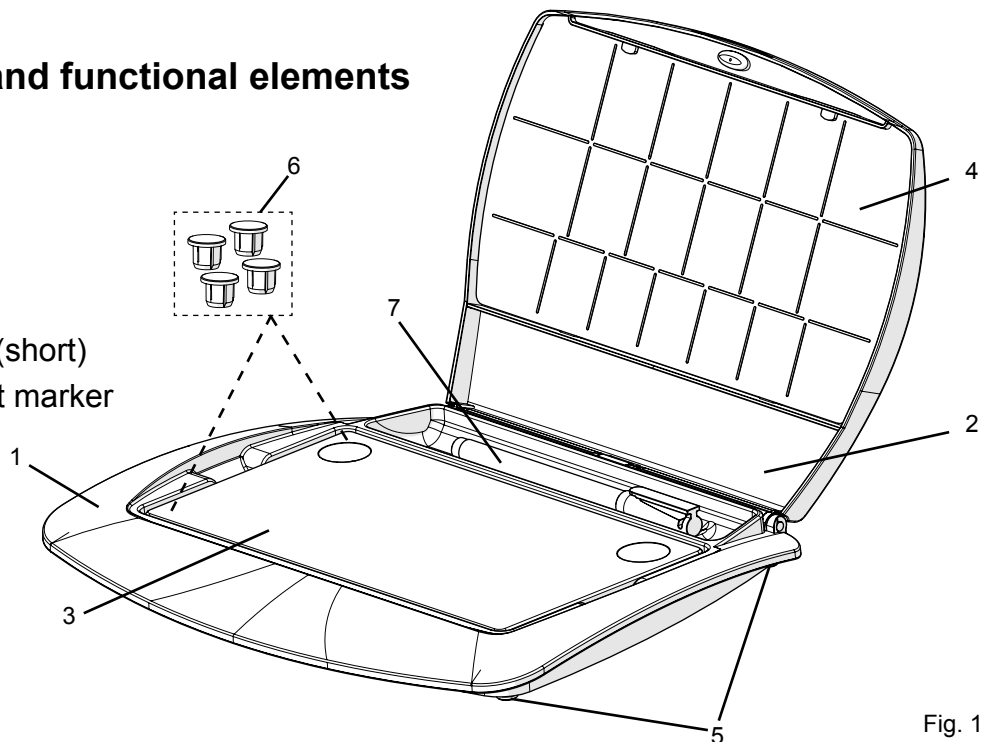
Fig. 1

2. Functional description of the case

Depending on your choice of mixing palette, you will either receive a large case with integrated tool compartment or a small case. When the lid is opened in a 90° angle it can be easily removed from the case base. Gently press the onto the base (1) and pull the lid (2) upwards.

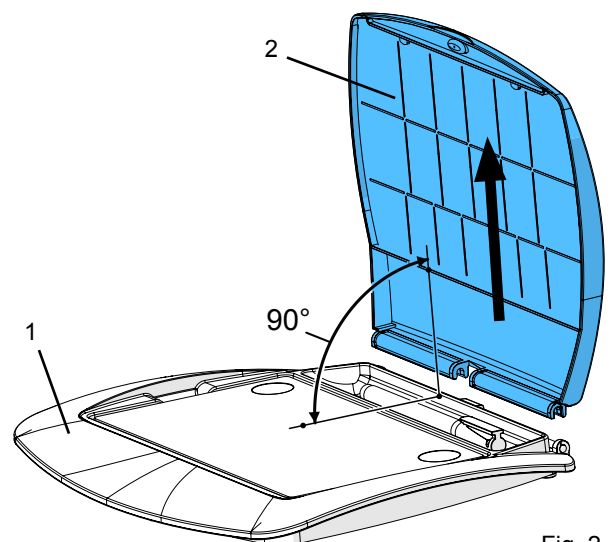
Fig. 2

At an angle of 115° there is a lid locking position hold the lid open in a stable position.

Caution:
If this position is overstretched, the hinges could become damaged.

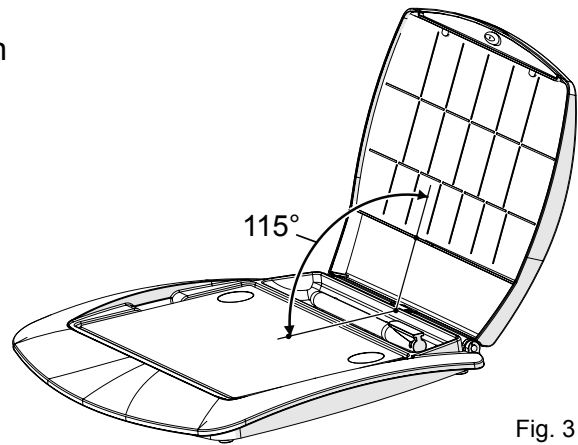
Fig. 3

3. Labelling foil

The non-permanent marker pen provided (7) can be used to write on the labelling foil (4). The foil can be removed from the lid in order to exchange or inscribe the label. Use the non-reflective side of the foil to write on.

When replacing the foil, clip behind the 4 mounting points which can be found on the lid.

Firstly, bend the retaining flaps towards the matt surface (a).

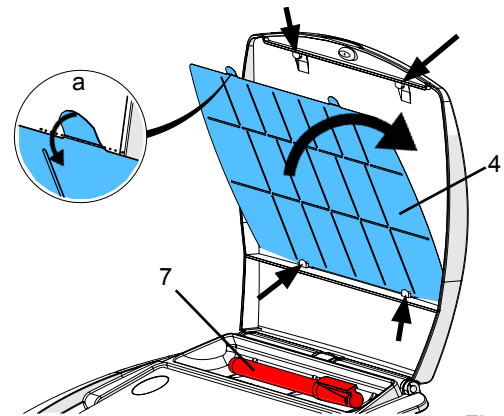
Fig. 4

4. Preparation for use

- After unpacking, quickly blow with compressed air.
- Lay the palette into the case base.
- Fill **exclusively distilled water** into the holes at the side of the base. Tap water will have a detrimental effect on the function and will lead to calcification of the capillaries.
- The maximum fill level is approx. 4 mm below the upper edge of the porcelain mixing palette.
- As soon as you notice an even distribution of moisture on the surface of the mixing palette, it is ready for use. On initial use it will take approx. 1 hour for complete saturation with water.

5. Use

Mixing with modelling liquid:

Place the required amount of porcelain powder onto the porcelain mixing palette surface. Add sufficient modelling liquid immediately until the desired saturation level is achieved. This porcelain consistency will be maintained for the next few days through the capillary effect of the palette.

Mixing without modelling liquid:

Place the required amount of porcelain powder onto the porcelain mixing palette surface. The capillary action of the palette will keep the porcelain moist until the optimal saturation level is achieved. Simply mix the porcelain gently through and it will be usable for several days at the same consistency.

Note:

The water level will decrease in time, it is therefore necessary to refill distilled water into the case base occasionally. The glazed, impermeable wells are designed to hold modelling liquid, shade mixes etc.

Never mix the porcelain materials on the porcelain mixing palette using a metal spatula. Instead, please use our agate spatula (No. 1157-0000). The metal abrasion can cause a risk of discolouration of the mixing palette surface and the porcelain itself!

6. Care instructions

In order to maintain the consistent quality of the mixing palette, it is essential to clean on a regular basis.

Clean carefully with a steam cleaner or brush using distilled water.

Clean the case using conventional detergent and then rinse thoroughly under running water.

Do not clean using sharp edged, abrasive or solvent based materials. Please only use soft brushes or cleaning cloths.

When using Tricolor stains it may happen that vaporized stains fluid etches the plastic lid.

Do not clean the plastic lid with monomers or with liquids containing solvents!

In time, the capillary action in the porcelain mixing palette will gradually decrease due to impurities in the water and dust from the environment. In addition to this, colour deposits from the porcelain powders may accumulate on the surface. By firing the porcelain mixing palette in a preheating furnace it can be made to look as good as new again:

- Take the porcelain mixing palette out of its case and remove all porcelain residues.

Allow the palette to dry for approx. 3 - 4 hrs. Place the dry porcelain mixing palette into the **cold** preheating furnace a gentle procedure is guaranteed if firing wool is placed underneath eg. Fibertray (No. 1919-000).

Do not place any pressure onto the porcelain mixing palette, and fire only one at a time!

Heat up the preheating furnace at a max.heating rate of 7 – 9 °C/min until 120 °C (248 °F) and hold at this temp. for 60 min. Then increase the heat at a rate of max. 6 – 8 °C/min to the final temperature of 800 °C (1,472 °F).

- Once the preheating furnace has reached it's final temperature, switch it off and allow to cool down with the door closed. Once the furnace is completely cool, remove the porcelain mixing palette.

The porcelain mixing palette should be cool enough to touch. Only remove from the furnace with your hands.

Do not use metal tweezers or investment ring tongs.

- Rinse the completely cooled porcelain mixing palette with distilled water in order to remove any potential combustion residues. Polish with 600 grit, wet abrasive paper and follow the instructions in point no. 4 (preparation for use).

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Renfert GmbH • Industriegebiet • 78247 Hilzingen/Germany