by Michel Magne

NEW OUTLINE

MANUAL

NEW OUTLINE

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Ihr anaxdent-Team

NEW OUTLINE

The tooth-colored methyl methacrylate NEW OUTLINE.

NEW OUTLINE is the ideal PMMA material to make mock-ups and durable temporaries.

Beautiful, rich, color-fast and dense dentins help to cover framework materials and provide a safe color background. A mixture of incisors with high/medium/low values and Transpa Clear helps you to individually control the brightness value in the incisal area.

The excellent material features ensure a durable stability. Its high flexural strength and good modulus of elasticity reduce the fracture susceptibility of the restorations.

Thanks to its high density and to its homogeneity, NEW OUTLINE can be considered as extremely gum-friendly. Thus, the material can be polished brightly. A great mixing ratio, a thin viscosity, and easy polishing (as mentioned before) make NEW OUTLINE top material for the user.

In addition to the dentins, two opaque dentins are available to make highly esthetic restorations. Thinly applied, they help to cover frameworks or discolored dies.

The combination with Skin Primer, Bonding Fluid, or A+P Bonder ensures a safe bonding to other materials (diacrylate, composite, acetal, polyamide).

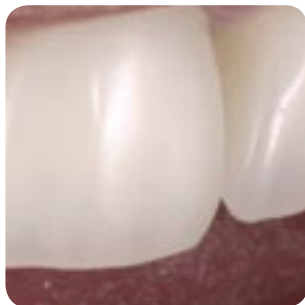
The indication includes, besides making mock-ups and temporaries, also the completion of prosthetic cases, repairs of PMMA acrylic, and the fixation of acrylic teeth. Within the last decade, NEW OUTLINE has been constantly improved in close collaboration with dental technicians and dental-care providers.

NEW OUTLINE - an easy-to-use but highly esthetic material

Please give us a call. We would be pleased to inform you more detailed.

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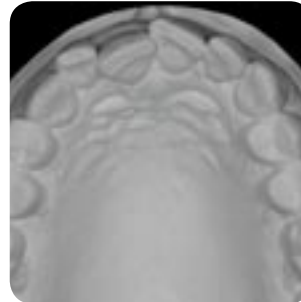
THE „MOCK-UP“

... is a quick and effective realization of the wax-up in the patient's mouth and is understood as demonstration and means of motivation for the patient to show how a definite solution could look like. To explain this step to the patient, you can tell him/her that a mock-up serves as „test run“ or as „conceptual test“ or „prototype“.

Duplicate the wax-up, pour New Outline acrylic into the matrix, and reposition it on the model treated with a separating agent. Just apply Skin Glaze (surface sealant) - and the mock-up is ready to be fixed in the patient's mouth for a short period of time.

The idea behind this method is that the situation, and thus the definite restoration, can be determined together with the patient by means of the mock-up and the following temporaries.

The changes made at the restoration during these steps are easy, quick and recurrent processes. Thus, shape and function can be determined at once.

When transferring the situation with silicone matrix or in the anaxform flask, which means a more accurate precision, all steps and situations can be performed within no time and are reproducible at any time.

Due to the internal characterization, color effects in temporaries can be added without any problems as well as later corrections, supplementations or reductions of the restoration.

Using New Outline or Visco (bisacrylic), the dental-care provider can work out pontic design and gingiva management directly in the patient's mouth.

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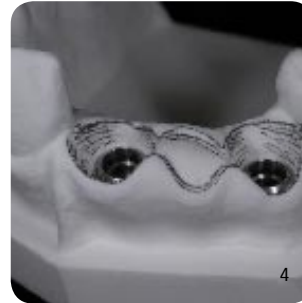
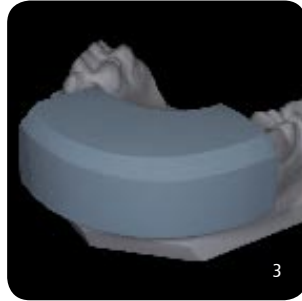
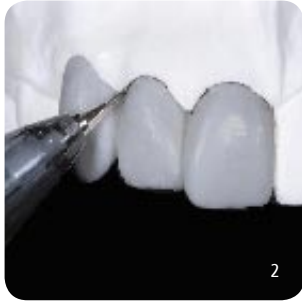
MODEL PREPARATION

Lab-made temporaries in sandwich technique

Fig. 1, 2, 3, and 4

Wax-up on a duplicate model; determination of the tooth volume and of the emerging profile

Version 1: Make a matrix in duplicate (matrix form) with silicone putty (let set in a pressure pot at 4 bars).

Fig. 5

Version 2: Duplicate the model in the anaxform flask.

Fig. 6, 7

Checking the matrix with temporary abutments screwed in

Fig. 8, 9, and 10

Mark the matrix margin with a pencil. Cover the temporary abutments with anaxblend Opaque, polymerize. Water the model and insulate it carefully with Separator.

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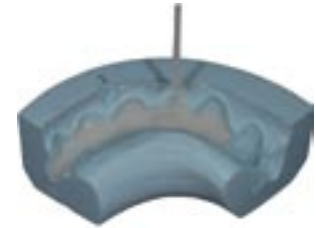
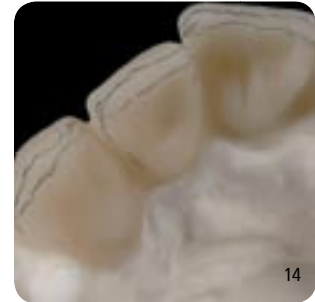
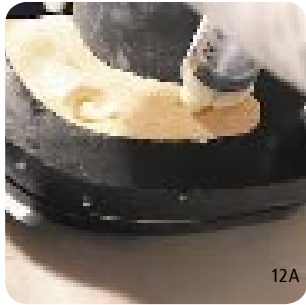
DENTIN DESIGN

Fig. 11 and 12

Mix New Outline powder and liquid in the ratio 2:1. Fill New Outline into the sulcus without any bubbles.

Version 1: Pour the creamy consistency into the rim. Reposition the rim on the model; let excessive material drain. Fix the rim with rubber bands at the model while aligning at the retentions in the silicone.

Version 2: Aspirate New Outline into a syringe and inject it into the anaxform flask.

Polymerize in a pressure pot (water of approx. 40°C) at 2 bars. The polymerization takes 10 minutes. Then, remove rim and excessive material.

Fig. 13, 14, and 15

Cut the second rim into 2 parts and mark the dentin core (fig.14). Remove the enamel layer (fig.15) and check the reduction with the rim. Thinly phase out the dentin in the incisal area.

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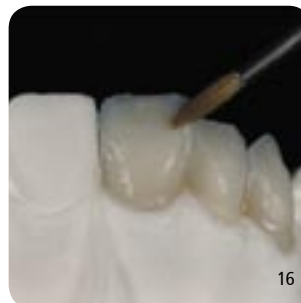
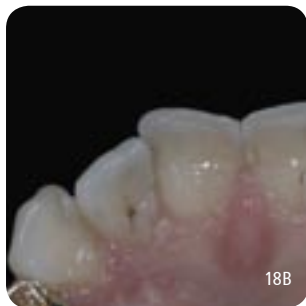
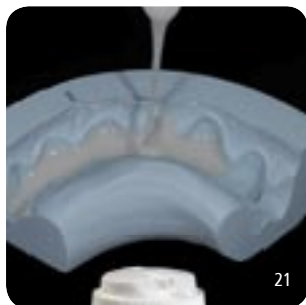
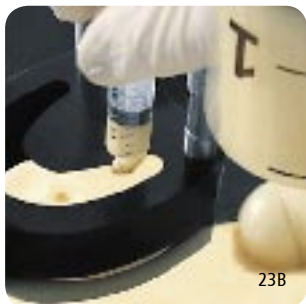
INTERNAL COLORATION

Fig. 16

Apply and polymerize Skin Primer to condition the surface.

Fig. 17, 18A, and 19

The incisal area and the palatal surfaces are mixed with anaxblend colors (with Clear), individualized, and light-cured (fig.18B).

Fig. 20

Apply and polymerize Skin Primer to condition the surface (bonding) for the enamels.

Fig. 21, 22 and 23A

High-, medium- or low value enamels mixed with Transpa Clear - and in

Version 1: pour it into the rim in a runny consistency. Reposition the matrix on the model. Let excessive material drain. Use rubber bands to fix the matrix on the model.

Fig. 23B

Version 2: Aspirate into the syringe and inject into the anaxform flask. Polymerize in a pressure pot (water of approx. 40°C) at 2 bars.

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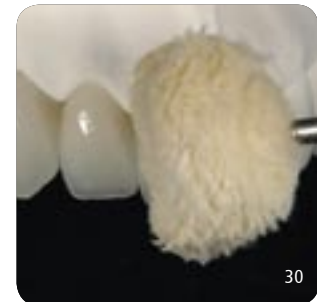
SHAPE, SURFACE, AND POLISH

Fig.24, 25, and 26

Prepare shape and function on the model.

Fig.27, 28, and 29

Mark the marginal ridges at the incisal run and at the interproximal space according to the run of the root or implant. Process surface and texture. The margins are prepared on the die. Optimize the margins on the second die.

Fig.30

Polish at a slowly running polishing motor (1,500 rpm) using a short brush, pumice powder or polishing paste.

Fig.31

For an intense characterization of temporaries, you can mix anaxblend colors stains into Skin Glaze and apply the mixture afterwards. Skin Glaze is a light-curing material with a polymerization time of 2 minutes per side with a plasma lamp or 4 minutes per side with a halogen lamp. Optimum bonding of Skin Glaze is ensured if you apply Skin Primer on the surface of the temporary.

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deutsch

Bilder: Michel Magne

Provisionals powered by:
Dr. Milko Villarreal, Murilo Calgato CDT. - Brazil

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