


# Evetric® Bulk Fill | Evetric® Bulk Flow

1

Self-Etch




OR Etch & Rinse

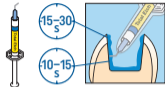

OR

Selective Enamel-Etch

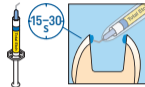

2

Evetric® Bond



3

Optional

Evetric® BulkFlow



4

Evetric® BulkFill



Bluephase® PowerCure



For permanent teeth only  
For deciduous teeth see instructions for use

	Evetric Bulk Fill/ Evetric BulkFlow	Evetric Bond
500 – 1,400 mW/cm <sup>2</sup>	–	10 s
500 – 900 mW/cm <sup>2</sup>	20 s	–
1,000 – 1,300 mW/cm <sup>2</sup>	10 s	–
1,800 – 2,200 mW/cm <sup>2</sup>	5 s	–

Date information prepared: 2024-02-14  
765481 / Rev. 0

ivoclar

see  
Instructions  
[www.ivoclar.com/eIFU](http://www.ivoclar.com/eIFU)

Rx ONLY  
For dental use only!


CE 0123

# Evetric® Line

## 4-mm composites

SCULPTABLE  
Evetric® Bulk Fill

FLOWABLE  
Evetric® Bulk Flow

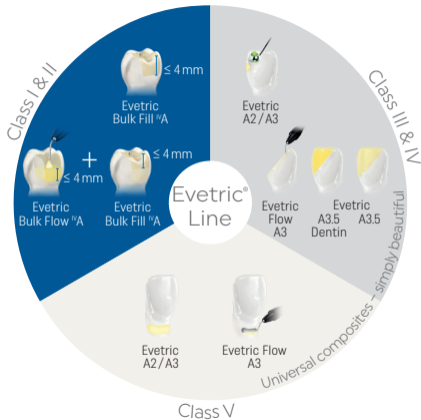

## Conventional composites

SCULPTABLE  
Evetric®

FLOWABLE  
Evetric® Flow