CanalPro™ CL2 Cordless Handpiece

User's Guide

- EN** User's Guide
- DE** Gebrauchsanweisung
- ES** Manual del usuario
- FR** Notice d'utilisation
- IT** Manuale per l'utente
- ZH** 用户指南

Table Of Contents

1.0	Product Overview	4
1.1	Indications for Use	4
1.2	Product Classification	4
1.3	Precaution for Safe handling and Operation	4
2.0	Safety Precautions	5
3.0	Product Components	6
3.1	Controls and Indicators	7
3.2	Charging Base	7
4.0	Operation	8
4.1	Switches	8
4.1.1	Power Switch	8
4.1.2	Motor On/Off Switch	8
4.1.3	Program Switch	8
4.1.4	Select Switch	8
4.1.5	Increment/Decrement Switches	9
4.2	Charging the Battery	9
4.3	The CanalPro™ CL2 Modes	10
4.4	Program Memory Function	11
4.5	Attaching and Removing the Contra Angle	11
4.6	Insert or Remove NiTi File	12
4.7	Buzzer Setting	12
5.0	Error Messages	13
6.0	Cleaning, Maintenance and Repair	13
6.1	Maintenance and Repair of Contra Angle	13
6.2	Cleaning Procedure	14
6.3	Changing the Battery	15
7.0	Troubleshooting	16
8.0	Product Specifications	17
9.0	Warranty Information	18

1.0 Product Overview

1.1 Indications for Use

This application area extends to endodontic procedures using a root canal instrument which is intended by the manufacturer for use in the mechanical and rotary preparation of root canals.

 CAUTION!	Federal law restricts this device to sale by or on the order of a healthcare practitioner
--	---

1.2 Product Classification

- Type of protection against electrical shock: Class 2 Equipment
- Degree of protection against electrical shock: Type B Applied Part. 
- Not suitable for use in the presence of a flammable anesthetic mixture with air, oxygen or nitric oxide.
- Continuous operation

1.3 Precaution for Safe handling and Operation

- Read and understand the ‘Safety Precaution’ prior to use to help use the product safely.

Classification	Degree of Danger, Damage or Seriousness
 DANGER!	Indicates a potentially hazardous situation which could result in serious injury or damage to the device.
 WARNING!	Indicates a potentially hazardous situation which could result in minor or moderate injury or damage to the device.
 CAUTION!	Indicates instructions to be observed for safety.

2.0 Safety Precautions


 <p>WARNING!</p>	<ul style="list-style-type: none"> • The system may be subject to malfunction when used in the presence of electromagnetic interference. Do not locate the system in the vicinity of other devices that possibly emit electromagnetic interference. • CanalPro™ CL2 is designed for use by dental professionals only. • Special care must be taken to maintain patient safety while in use. • Read this user's guide thoroughly and understand functions of individual components completely before use to help avoid undesired operation. • Ensure that the CanalPro™ CL2 is in good working condition prior to use to avoid undesirable operation from occurring. • Before use perform a test run to ensure the system is working properly to avoid undesirable operation from occurring. • If any abnormal condition is detected including excessive vibration, noise and heat while in use quit using the product immediately and contact customer service. Use under abnormal conditions may result in injury. • Ensure the handpiece is OFF before installing or removing a file to avoid injury from occurring. • Do not lubricate the handpiece. This may result in improper operation. • Both the handpiece and charging base should not be sterilized. Equipment may be damaged by the sterilization process. • Do not use organic solvents to clean handpiece or charging base. Organic solvents may cause damage to the equipment.
 <p>CAUTION!</p>	<ul style="list-style-type: none"> • For follow-up service or spare parts contact customer service. • Do not expose the product to dust, sulfur or salt to avoid equipment damage. • Use recommended (rotary) files only. Use of non-recommended files may lead to undesirable results.

3.0 Product Components

<p>Charging Base (60018733)</p>	
<p>Handpiece (60018730)</p>	
<p>Contra Angle (16:1) (60018731)</p>	
<p>Handpiece Cap (60018735)</p>	
<p>Oil Guide (60018736)</p>	
<p>Power Cord (EU) (60019371)</p>	
<p>Power Cord (60018734)</p>	

3.1 Controls and Indicators

Change in Symbols			
Alarm Indicator	Battery Indicator	Mode Indicator	Auto Stop Reverse Forward
On	Full		Auto Stop Reverse Stop
Off No Indication	25-75%		Auto Stop Reverse Stop
	25% or less	No Indication	Auto Stop
	Empty		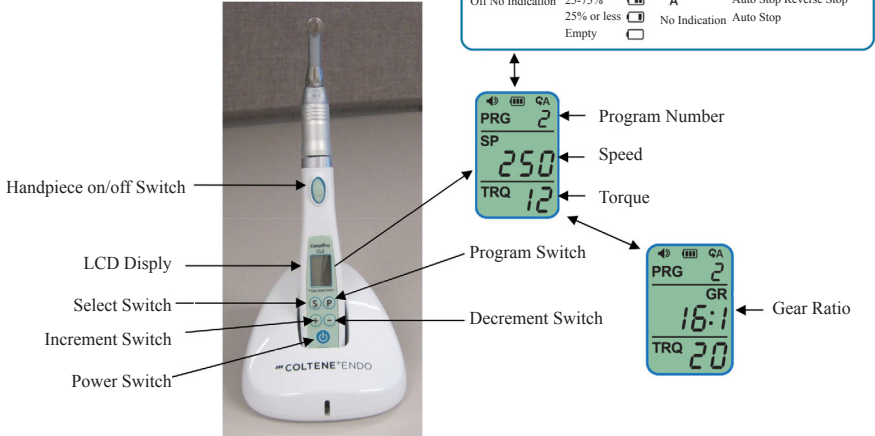

Handpiece on/off Switch

LCD Display

Select Switch

Increment Switch

Power Switch

Program Number

Speed


Torque

Program Switch

Decrement Switch

Gear Ratio

3.2 Charging Base

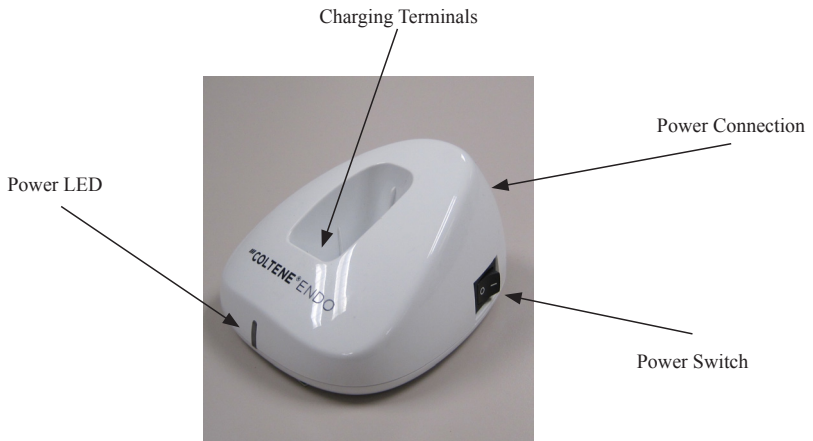Power LED

Charging Terminals

Power Connection

Power Switch

4.0 Operation

4.1 Switches

4.1.1 Power Switch

4.1.1.1 To turn it ON, press and hold the switch for longer than 1 second when the power is OFF.

4.1.1.2 To turn it OFF, press and hold the switch for longer than 1 second while the power is ON.

4.1.2 Motor On/Off Switch

4.1.2.1 Simply pressing this switch will turn the motor on or off depending on its current state (ON or OFF).

4.1.3 Program Switch

4.1.3.1 Toggles through mode 1, 2, 3 and 4 for each time the switch is pressed. The preset values (speed, torque and gear ratio) are displayed as each mode is selected.

4.1.4 Select Switch

4.1.4.1 Pressing this switch will change the menu item selected. The operation of this switch is dependent on whether the handpiece is on or off.

4.1.4.1.1 If the handpiece is off this switch will toggle between Speed, Torque, Gear Ratio, Run Mode and Speed.

4.1.4.1.2 If the handpiece is on this switch will toggle between Speed and Torque.

	Menu Item Descriptions	
Speed (SP)	—	Speed can be adjusted by pressing the increment or decrement switch.
Torque (TRQ)	—	The torque of the handpiece is adjustable.
Gear Ratio (GR)	—	Sets the gear ratio of the contra angle which is connected to the unit.
Run Mode (PRG)	—	Selects the mode of the handpiece.

- 4.1.4.1.3 When Auto Stop Reverse Forward (ASRF) is selected: **G** and **A** will blink.
- 4.1.4.1.4 When Auto Stop Reverse Stop (ASRS) is selected: **A** blinks
- 4.1.4.1.5 When Auto Stop is selected: **G** blinks
- 4.1.4.1.6 The select switch will be initialized (speed adjustment mode) if the program switch is pressed while selecting any item using the select switch.

4.1.5 Increment/Decrement Switches

- 4.1.5.1 The switches are used to increase (Increment Switch) or decrease (Decrement Switch) the value of the item selected using the select switch.

4.2 Charging the Battery

Prior to operation the battery must be fully charged according the instructions below:

- 4.2.1 Ensure that the site main power is working and that the CanalPro™ CL2 is corded with the proper type of power cord for power available at the site.
- 4.2.2 Plug the power cord into both the charging base and the wall outlet.
- 4.2.3 Turn the power switch on the charging base to on and ensure that the charging LED lights green.
- 4.2.4 Place the handpiece into the charging base before the first use and fully charge the handpiece. The charging base LED will be yellow when the handpiece is not in place. When the handpiece is place in the charging base the LED will light green indicating that the unit is charging. An audible buzzer will sound when the handpiece is placed in the charging base.

WARNING!

If there is not audible buzzer or LED light on after the handpiece is placed in the charging base immediately stop using the CanalPro™ CL2 and contact Customer Service.

Note the following:

- If the charging LED is not lit after the power is turned on, there may be a problem with the base circuit. Contact Customer Service for instructions.
- If the charging base terminals are dirty and contaminated, the handpiece may not properly charge. Ensure all connections are clean prior to using the charging base.
- Use only the charging base supplied with the CanalPro™ CL2.
- While the battery is charging the rear of the handpiece may become warm which will not generally cause a problem.
- Once the handpiece is placed on the charging base it is recommended that it stays in place until the battery indicates fully charged.
- A completely drained battery may not be able to be charged and may have to be replaced.
- Avoid placing the charging base in an area that may be susceptible to additional heat (in the sunlight or next to a heater, etc.).
- The battery may not charge if the temperature is excessively high or low, it is already fully charged or if the battery is damaged.

4.3 The CanalPro™ CL2 Modes

- Auto Stop Reverse Forward  – In this mode if the handpiece is overloaded it will begin to run in reverse until the overload is relieved.

It will then begin to run in forward mode.

- Auto Stop Reverse Stop **A** – In this mode if the handpiece is overloaded, it will begin to run in reverse. Once the overload is relieved the handpiece shuts off.
- No Indication – Auto Stop – In this mode when the handpiece is overloaded it turns off.

 WARNING!	<p>If the battery charge is too low, the actual load may not reach the selected torque value, in which case the Auto Reverse function will not work.</p> <p>If the battery is overheated due to use an error message will be displayed. It is necessary to allow the handpiece to cool at this point.</p>
--	---


4.4 Program Memory Function

- 4.4.1 **P** The unit has four memory functions which will store the user's own speed, torque, gear ratio and operational mode. These modes can be toggled through by pressing the Program button.

4.5 Attaching and Removing the Contra Angle


- 4.5.1 The Contra Angle simply snaps on or off to the top of the handpiece. A 16:1 gear ratio contra angle is supplied with the CanalPro™ CL2 and additional contra angles can be ordered. The CanalPro™ CL2 will also operate with Contra Angles of other gear ratios. Contact your local sales operation for more details.

 WARNING!	<p>DO NOT use any contra angle attachment from other manufacturers with this unit. Failure to comply may result in equipment damage.</p> <p>Ensure the handpiece power is off before removing or replacing the Contra Angle. Ensure the Contra Angle is securely installed prior to use. Failure to comply may result in equipment damage.</p>
--	--

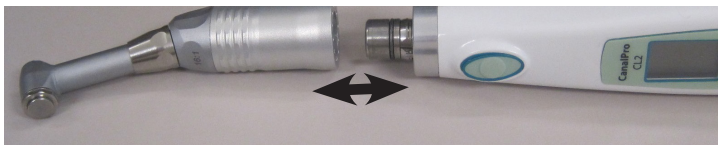4.6 Insert or Remove NiTi File

- 4.6.1 While pressing and holding the the button on the back of the the contra angle, lightly rotate and insert the file.
- 4.6.2 Ensure the file is correctly installed by lightly pulling on the file to ensure it does not come loose.
- 4.6.3 A file is removed by pressing the button and pulling the file while the button is pressed.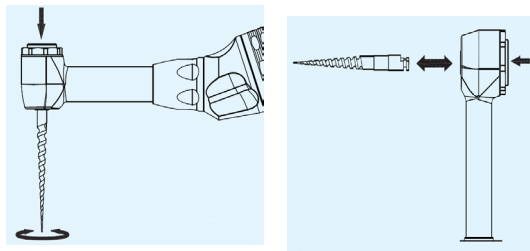

 CAUTION!	<ul style="list-style-type: none"> • Ensure the power is OFF before inserting/removing a file. • Check that the file is properly installed by lightly pulling on the file. • Clean the file prior to inserting to avoid improper insertion. • Do not exceed the recommended RPM for the file in use. • Failure to comply with these Cautions may result in equipment damage or injury to personnel.
--	---

4.7 Buzzer Setting

- 4.7.1 The buzzer can be set for On/Off when the preset torque reading has been reached.
- 4.7.1.1 The buzzer select menu is displayed by pressing the power switch while pressing and holding the select switch after turning the power Off. When the buzzer indicator is displayed, a warning sound is made unless the buzzer is set to Off.
- 4.7.1.2 The warning sound can be alternately selected (on/off) by pressing the select switch while pressing and holding the power switch.
- 4.7.1.3 The warning on/off selection can be stored by releasing the power switch and can be used for general conditions.

5.0 Error Messages

Error codes are displayed on the LCD and are described below.

Note

Errors are cleared by pressing the handpiece On/Off switch. The unit will then return to standby mode.


Error Code	Error	Cause	Solution
E1	The preset torque has been exceeded.	<ol style="list-style-type: none"> The handpiece is locked. The load is exceeding the preset torque. 	<ol style="list-style-type: none"> Check if the angled portion is locked. Increase the preset torque value.
E2	Battery Voltage is Excessively high.	<ol style="list-style-type: none"> Battery type is incorrect The circuit is defective. 	<ol style="list-style-type: none"> Replace the battery. Contact customer service.
E3	Battery Current Error is detected.	Current is excessively low even in the Run mode or when the current is excessively high during operation.	Contact customer service.
E4	Battery voltage too low while charging.	Voltage is low within 10 minutes after charging.	Replace the battery.
E5	Battery voltage too high while charging.	Voltage is high 10 minutes after charging.	Replace the battery. Contact customer service.
E6	Battery Temperature is excessively high	Battery temperature abnormally high.	Replace the battery. Contact customer service.
E7	Motor Driver Circuit is Defective	Circuit board damaged.	Contact customer service.

6.0 Cleaning, Maintenance and Repair

6.1 Maintenance and Repair of Contra Angle

The Contra Angle is the only CanalPro™ CL2 attachment that can be lubricated. Lubrication is required prior to each use as well as before sterilization.

- 6.1.1 Lubricate once a day prior to sterilization. Insert the spray nozzle into the end of the contra angle that attaches to the handpiece. Spray the lubricant 5 times.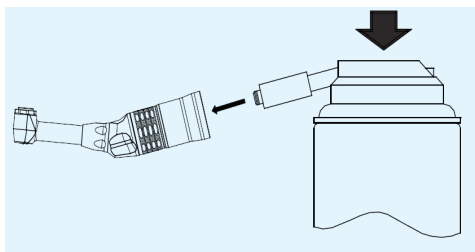

- 6.1.2 To remove dirt and contamination from the contra angle, wipe with a soft cloth saturated with rubbing alcohol.
- 6.1.3 When the contra angle is detached to perform lubrication or sterilization and charge the handpiece place the cap on the handpiece to prevent it from becoming contaminated. Use a standard motor handpiece lubricant.

**CAUTION!**

- Never lubricate the handpiece. Failure to comply may result in equipment damage.
- Completely wipe off oil from the contra angle before connecting it to the handpiece. Also do not attach the contra angle to the handpiece until the oil has dried. Failure to comply may result in equipment damage.
- It is recommended to hold the spray can vertically.
- Do not use soluble liquids such as benzene or thinner when cleaning the handpiece. Failure to comply may result in equipment damage.
- Never autoclave the handpiece. Failure to comply may result in equipment damage.

6.2 Cleaning Procedure

- 6.2.1 Clean the contra angle using a soft brush and then wipe out the dirt on its surface with a soft cloth (do not use a wire brush).
- 6.2.2 Lubricate the contra angle as described in section 6.1.
- 6.2.3 Place the contra angle into the autoclave.
- 6.2.4 Set the autoclave temperature to approximately 121° C (250° F) for approximately 20 minutes and then 132° C (270° F) for approximately 15 minutes.
- 6.2.5 When complete allow the contra angle to dry and keep it in a clean area.


6.3 Changing the Battery

The handpiece battery is rechargeable over 500 times depending on the user's operational methods and the handpiece conditions. If the handpiece operational time or RPM has been considerably reduced, the battery should be considered bad and replaced. Contact your dealer for a battery replacement. Observe the following precaution when replacing the battery.


- Only open the battery cover.
- Ensure that only the designated battery (AAA nickel metal hydrogen rechargeable battery, 1.2 V, standard) is used. Specifications vary depending on the manufacturer. Using the incorrect batteries may result in equipment damage.
- Do not use non-rechargeable batteries (*alkaline or manganese*). This may result in an explosion and gas emission during charging.
- If the liquid leaks or direct contact with the eyes occurs immediately rinse thoroughly with water and seek medical attention.
- Replace only with the correctly designated batteries and replace both batteries at once. Shortened battery file may result if both batteries are not replaced at the same time.
- Do not replace batteries with wet hands to avoid contamination to the battery terminals.
- Dispose of used batteries in accordance with the local ordinances.

6.3.1 Turn the power off.

6.3.2 Remove the rubber cover and cover screw.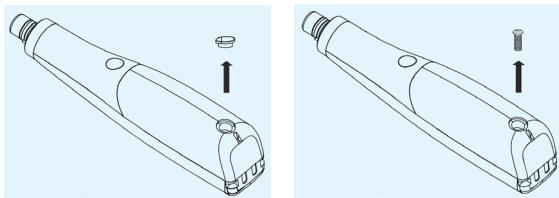

6.3.3 Remove the battery cover.

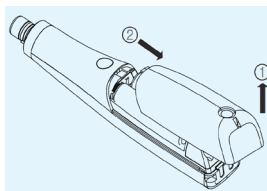6.3.4 Remove the old batteries and replace with the new batteries using care to observe the correct polarity.

6.3.5 Re-install the cover (using hardware previously removed). Avoid over tightening the screw. Place the unit on the charger until it indicates that the battery is fully charged.

7.0 Troubleshooting

Problem	Cause	Solution
LED Does not turn on.	<ol style="list-style-type: none"> 1. Battery completely drained 2. Handpiece not used for an extended period of time. 	If the battery is not completely drained, it must be recharged. <i>If it is completely drained it must be replaced.</i>
Battery not properly charging	<ol style="list-style-type: none"> 1. Battery completely drained 2. Battery temp too low. 3. Battery temp too high. 4. Battery voltage is too high. 5. Improper contact with charging base. 6. Error code is displayed 	<ol style="list-style-type: none"> 1. Replace the battery 2. Place the charger in an area with proper temperature control above 0° C (32° F). 3. If the battery temperature is too high and the battery will not charge, contact customer service. 4. Ensure the correct batteries are being used. 5. Ensure that the handpiece is properly mounted to the charging base. 6. Check the error message.
Charging base is not working	<ol style="list-style-type: none"> 1. Power cord not plugged in. 2. Power is switched Off. 3. Blown Fuse. 	<ol style="list-style-type: none"> 1. Check power connections 2. Turn power switch on. 3. Contact customer service.
Handpiece overheating on charging base	Battery or charging base is defective.	Contact your local dealer.
Handpiece head not rotating	Debris in motor head	Clean or replace the contra angle.
Handpiece power weak	Battery is low	Charge the battery
Auto/Reverse function not working	Battery is low	Charge the battery
RPM of handpiece slow	Battery is low	Charge the battery

8.0 Product Specifications

Handpiece

- Battery Type: Nickel Metal Hydrogen (AAA Size)
- Input Voltage: 2.4V
- Rated Capacity: 1000mAH
- Dimensions: 135 (W) x 27 (D) x 28 (FH)
- Weight: 150g

Charging Stand

- Input Power: 100/240 VAC
- Input Voltage: 6VA
- Rated Current: 800mA
- Output: 135 (W) x 27 (D) x 28 (H)
- Weight: 160g

Contra Angle

- Gear Ratio: 4:1, 10:1, 16:1 and 20:1

Transportation and Storage Requirements

- Temperature Limits -20°C (-4°F) - 60°C (140°F)
- Atmospheric Pressure Limits 700 - 1600 HPa
- Humidity Limits 0% - 90%

9.0 Warranty Information

Our products are carefully manufactured to meet stringent quality assurance requirements. Our products are manufactured from new parts or new and serviceable used parts. Regardless, our warranty terms apply. This product has been developed specifically for use in dentistry and is intended to be operated only by qualified dental professionals in accordance with the instructions contained in this guide. However, notwithstanding anything contained herein to the contrary, the user shall at all times be solely responsible for determining the suitability of the product for the intended purpose and the method of its use. Any guidance on application technology offered by or on behalf of the manufacturer, whether written, verbal or by demonstration, shall not relieve the dental professional of his/her obligation to control the product and to make all professional judgments regarding its use. Our products are warranted in accordance with the terms of a written Certificate of Limited Warranty accompanying each product. Except for the warranties specifically set forth in the Certificate of Limited Warranty, Coltène/Whaledent Inc. provides no warranties or guarantees of any kind covering the product, expressed or implied, including, without limitation, any warranties as to merchantability or fitness for a particular purpose. The purchaser/user is referred to the Certificate of Limited Warranty for all of the terms, conditions and limitations of the warranty covering this product. This Section of the user manual is not intended to in any way modify or add to the warranty provided in the Certificate of Limited Warranty. Any claim for damage or breakage to the product in transit should be made to the carrier promptly upon discovery.

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